


Ankle Support Figure 8

Our Ankle Support Figure 8 is constructed out of nylon that is durable and thin, so it can fit in a shoe and breathes easily. It uses a laces to strap Figure 8 design that provides effective compression to the ankle.

The straps are made of non-stretch nylon in order to limit movement to prevent further damage. There are also areas in the tongue and Achilles that have breathable and comfortable flex mesh material. Convenient and comfortable support that helps you feel better faster.

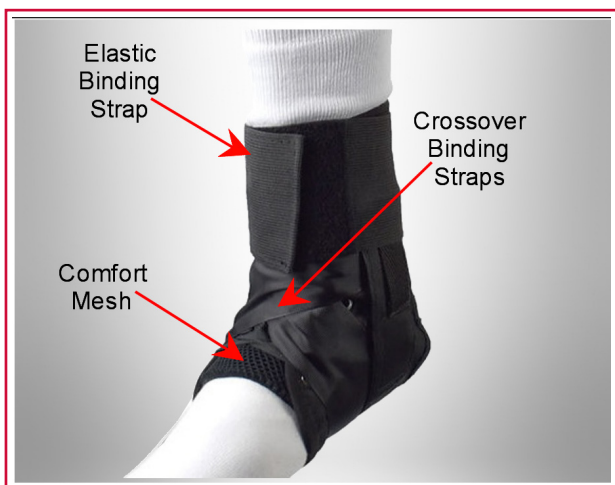

Design Highlights:

- Non-stretch material for comfort, performance and durability.
- Lace-up design and figure 8 strapping locks the calcaneus into place and controls inversion/eversion.
- Elastic cuff provides extra support and compression to help relieve ankle pain.
- Low profile so fits easily into most shoes.

Indications for Use:

- Mild Ankle Sprains
- Chronic Ankle Instability
- Post-Surgery Ankle Weakness
- Prophylactic Use

Ankle Support Figure 8

Fitting Instructions



Application



Step 1. Fully loosen straps and laces. Slide foot into brace under tongue.

Step 2. Place heel into opening.

Step 3. Starting closest to toes, tighten laces. Tie laces to secure.

Step 4. Wrap one figure-8 strap across laces, under foot and secure to strip on side of strap origination.

Step 5. Repeat with additional figure-8 strap.

Step 6. Wrap left ankle support strap across the front of foot, over laces.

Step 7. Secure right ankle support strap. Re-adjust as needed.

Care Instructions

- Hand wash with 50/50 alcohol/water mix
- Rinse and air dry
- Do NOT use bleach



All products are provided by TLC DME.

For more information regarding product application, functionality, and/or warranties, please call TLC DME Patient Care at 1.855.4TLC DME or email patientcare@tlcdme.com.

All DMEPOS products are covered by CMS, work comp, and commercial insurances. If you have not yet met the deductible associated with your insurance plan, you may have a financial responsibility.