

Wrist Orthosis

This Wrist Orthosis has been meticulously designed to offer the crucial support & comfort needed to enable your recovery, as outlined in your treatment plan from your provider. The lightweight breathable material combined with strategically customized support, will assist in alleviating pain and help you feel better faster.



Design Highlights:

- Easy slip-on application.
- Removable palmar stay is made of malleable aluminum for best custom support.
- Radial (top of wrist) stay is easily re-positioned for a proper fit.
- Low contour below palmer crease allows for unrestricted finger movement.

Indications for Use:

- Carpal Tunnel
- Soft Tissue Injury and Tendonitis
- Wrist Sprains and Strains
- Post-Cast Healing

Wrist Orthosis

Fitting Instructions



Application

Step 1. Position hand between the interior of the orthosis and underneath the stockinette lining.

Step 2. Ensure palmer stay (metal stay that supports the palm) is situated below middle crease on palm so there is full functionality of the fingers.

Step 3. Adjust radial stay (metal component on top forearm) by lifting off support and pulling away from body to tighten support. Stay should be centered on middle of forearm. Secure strap between thumb and forefinger.

Step 4. Adjust all straps as needed for the most comfortable fit.

1)



2)



3)



4)



Care Instructions

- Hand wash with mild detergent
- Rinse and air dry
- Do NOT use bleach



All products are provided by TLC DME.

For more information regarding product application, functionality, and/or warranties, please call TLC DME Patient Care at 1.855.4TLC DME or email patientcare@tlcdme.com.

All DMEPOS products are covered by CMS, work comp, and commercial insurances. If you have not yet met the deductible associated with your insurance plan, you may have a financial responsibility.